

MOUNTAIN MILLED DENTURES

Step by Step Protocol: Copy Denture - Analog to Digital

First Appointment

- ☐ Duplicate patient's existing dentures using a duplication jig
- ☐ Make final impression in duplicated dentures
- ☐ Take Bite Registration
- ☐ Patient Evaluation/Information Gathering

**3 Days in Lab to Fabricate
(design and print try-in)**

Second Appointment

- ☐ Printed Try-in
- ☐ Evaluate fit, form and function
- ☐ Contour teeth as needed to achieve ideal esthetics and occlusion
- ☐ Take new bite registration if needed
- ☐ Take wash impression inside of try-in if needed

**4 Days in Lab to Fabricate
(finalize design and mill
final denture)**

Third Appointment

- ☐ Delivery of Mountain Milled Denture
- ☐ Review Denture Home Care Instructions



First Appointment

- Duplicate patient's existing dentures using a duplication jig.



- Make final impression in duplicated dentures



- Take Bite Registration



☐ Patient Evaluation/ Information Gathering




- ☐ Evaluate Ridge/Palate/Vestibule for surgical considerations and advise patient of retention expectations



- ☐ Discuss Patient Goals for Smile, Mastication, Retention




- ☐ Evaluate Current VDO,
Select Shade

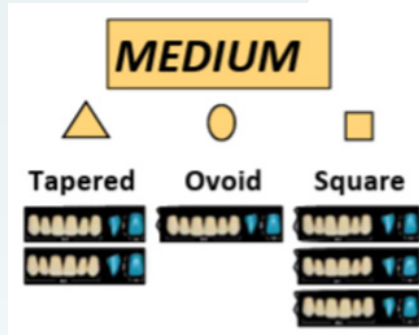

- ☐ Select Mould/Shape of Teeth - Note differences from existing



- ☐ Evaluate and note any desired changes to Occ Plane and Determine Type of Occlusion for new denture(s), (Balanced, Lingulized, Monoplane)

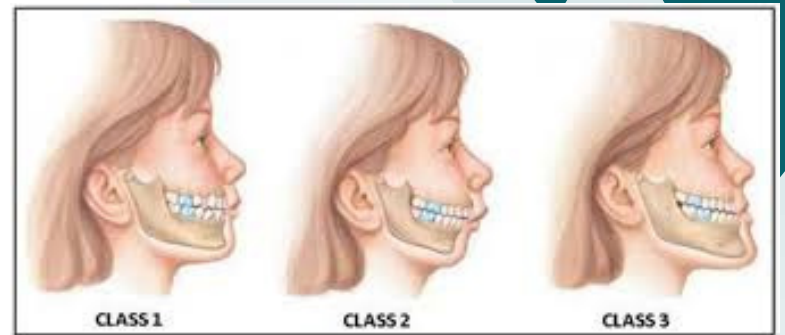

- ☐ Evaluate Current Occlusal Scheme/Classification



- ☐ Note Lip Line and any desired changes to existing

Second Appointment



- ☐ Printed Try-in



- ☐ Evaluate fit, form and function



- ☐ Contour teeth as needed to achieve ideal esthetics and occlusion



- ☐ Take new Bite Registration if needed



- ☐ Take wash impression inside of try-in if needed'

Third Appointment

☐ Delivery of Mountain Milled Dentures



☐ Review Denture Home Care Instructions

