

# Step by Step Protocol: Immediate Denture

## First Appointment

- ☐ Take Final Impression or Intraoral Scan
- ☐ Take Bite Registration\*
  - Additional Appointment may be Necessary
- ☐ Patient Evaluation/Information Gathering

4 Days in Lab to Fabricate  
(design and print try-in and  
custom tray)

## Optional Second Appointment (Esthetic)

- ☐ Printed Try-in
- ☐ Bite Registration if needed
- ☐ Final Impression with custom Tray

4 Days in Lab to Fabricate  
custom tray and  
occlusal rim.

## Optional Second Appointment (Functional)

- ☐ Final Impression with custom Tray
- ☐ Occlusal Rim Contouring/Bite Registration

3 Days in Lab to Fabricate  
(finalize design and mill  
final denture)

## Third Appointment

- ☐ Delivery
- ☐ Review Denture Home Care Instructions



# First Appointment

## ☐ Take Final Impression or Intraoral Scan



## ☐ Take Bite Registration



## ☐ Additional Appointment necessary if:

- ☐ Unable to take adequate impression and custom tray needed



## ☐ Patient Evaluation/Information Gathering


- ☐ Evaluate Ridge/Palate/Vestibule for surgical considerations and advise patient of retention expectations
- ☐ Discuss Patient Goals for Smile, Mastication, Retention




- ☐ Evaluate Current VDO, Select Shade



- ☐ Select Mould/Shape of Teeth - Note differences from existing

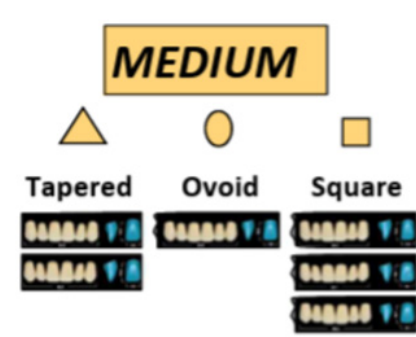

- ☐ Evaluate Current Occlusal Scheme/Classification

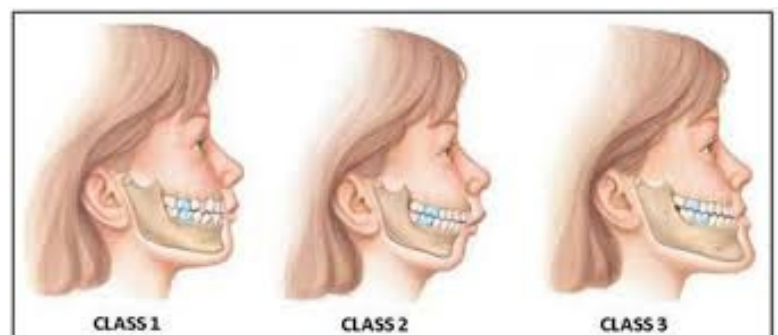

- ☐ Evaluate and note any desired changes to Occ Plane and Determine Type of Occlusion for new denture(s), (Balanced, Lingulized, Monoplane)



- ☐ Note Lip Line and any desired changes to existing



# Optional Second Appointment (Esthetic)

## ☐ Printed Try-in



## ☐ Take new Bite Registration if needed



## ☐ Final Impression with custom Tray



# Optional Second Appointment (Functional)

- ☐ **Final Impression with custom Tray**



- ☐ **Occlusal Rim Contouring/Bite Registration**



# Third Appointment

## ☐ Delivery of Mountain Milled Dentures



## ☐ Review Denture Home Care Instructions

